

Aumed[®]

SeeTV User Guide



Uses for your SeeTV

SeeTV binoculars are great for watching TV, but you can also use them for other activities such as sporting events, movies, theatre, bird watching, or any other distance viewing activity in which a magnified image would be helpful.

Viewing distance

The SeeTV is designed for viewing objects at a minimum distance of 10 feet (approximately 3 metres). You may have become used to watching TV at a closer distance than 10 feet, and you should now be able to sit further away.

Focusing

With your TV screen no nearer than 6 feet from you, close your left eye and adjust the right-hand focus wheel. Repeat for the other eye.

Care of your SeeTV binoculars

Avoid handling the lenses. To remove finger marks from the lenses use a clean soft cloth. If the lenses and frame become very dirty wash the SeeTV in warm, soapy water and rinse under the cold tap. Leave to dry naturally away from direct heat. Never use abrasive or solvent cleaners.

When not in use store your SeeTV in its protective pouch. Take care not to lay the binoculars down on the lens surface.

Pamtrad

The Stables, 22 Ruddington Lane, Wilford, Nottingham, NG11 7BH

Tel: 0115 9816636 email: info@pamtrad.co.uk